




1 Is my patient's fracture suitable for EXOGEN?

Fresh fracture EXOGEN accelerates fracture healing by 38%.^{1,2} **Delayed union** **Nonunion** EXOGEN heals 86% of nonunion fractures.³

2 Is the fracture personality associated with poor fracture healing?

YES

NO

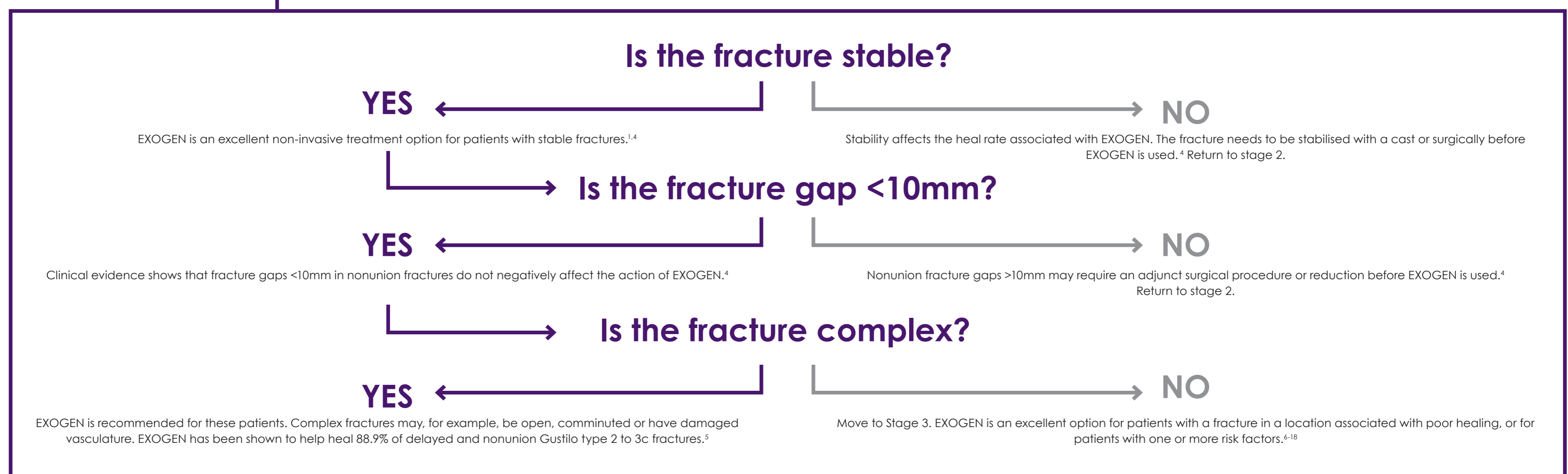


3 Does the patient have a fracture in a location associated with poor fracture healing?

YES

NO

EXOGEN could be helpful for these patients.

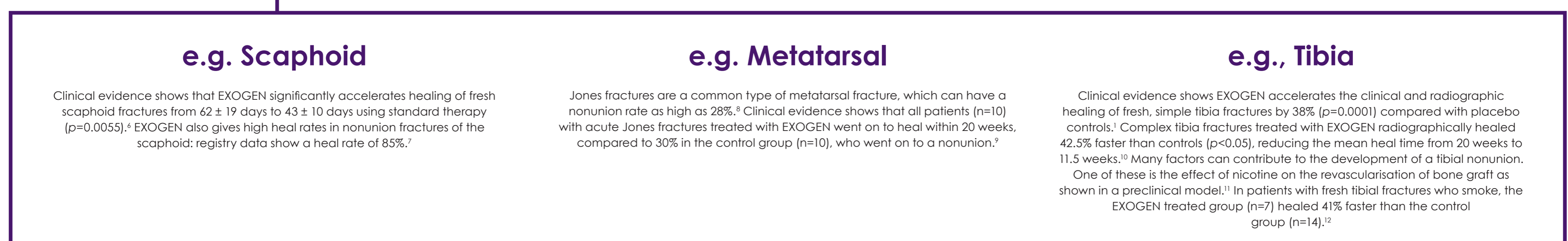


4 Does the patient have risk factors associated with poor fracture healing?

YES

NO

Patients with one or more of these risk factors are ideal candidates to be treated with EXOGEN.

EXOGEN is a viable treatment option for patients who need to heal faster. EXOGEN accelerates fresh fracture healing by 38%.^{1,2}

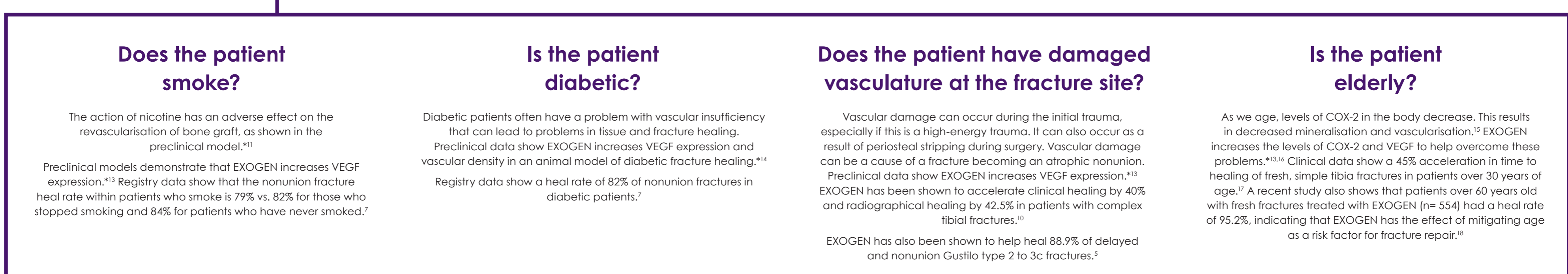

*The clinical relevance of in vivo findings is unknown.

References: 1. Heckman JD, Ryaby JP, McCabe J, Frey JJ, Killooye RF. Acceleration of tibial fracture-healing by non-invasive, low-intensity pulsed ultrasound. *J Bone Joint Surg Am.* 1994;76(1):26-34. doi:10.2106/0004623-199401000-00004 2. Kristiansen TK, Ryaby JP, McCabe J, Frey JJ, Roe LR. Accelerated healing of distal radial fractures with the use of specific, low-intensity ultrasound. A multicenter, prospective, randomized, double-blind, placebo-controlled study. *J Bone Joint Surg Am.* 1997;79(7):961-73. doi:10.2106/0004623-199707000-00002 3. Nolte PE, van der Krans A, Patka P, Janssen IMC, Ryaby JP, Albers GH. Low-intensity pulsed ultrasound in the treatment of nonunions. *J Trauma.* 2001;51(4):693-703. doi:10.1097/00005373-200110000-00012 4. Roussigne X, Currey C, Duparc F, Dujardin F. Indications and results for the Exogen® ultrasound system in the management of non-union: a 59-case pilot study. *Orthop Traumatol Surg Res.* 2012;98(2):205-13. doi:10.1016/j.otsr.2011.10.011 5. Lerner A, Stein H, Soudry M. Compound high-energy limb fractures with delayed union: our experience with adjuvant ultrasound stimulation (Exogen). *Ultrasounds.* 2004;42(1-9):915-7. doi:10.1016/j.ultras.2003.11.014 6. Mayr E, Rudzki MM, Rudzki M, Borchardt B, Häusser H, Rüter A. Does low-intensity pulsed ultrasound accelerate healing of scaphoid fractures? *Handchir Mikroschir Plast Chir.* 2000;32(2):115-22. doi:10.1055/s-2000-19253 7. Frankel VH, Mizuno K. Management of non-union with pulsed, low-intensity ultrasound therapy: international results. *Surg Technol Int.* 2002;10:195-200. 8. Rosenberg GA, Sterra JJ. Treatment strategies for acute fractures and nonunions of the proximal fifth metatarsal. *J Am Acad Orthop Surg.* 2000;8(5):332-8. doi:10.5435/00124635-200009000-00007 9. Teoh K, H., Whitham, R., Wong, J. F., & Hariharan, K. (2018). The use of low-intensity pulsed ultrasound in treating delayed union of fifth metatarsal fractures. *Foot (Edinburgh, Scotland)*, 35, 5255. https://doi.org/10.1016/j.foot.2018.01.004 10. Leung KS, Lee WS, Tsui HF, Liu PPL, Cheung WH. Complex tibial fracture outcomes following treatment with low-intensity pulsed ultrasound. *Ultrasound Med Biol.* 2004;30(3):389-95. doi:10.1016/j.ultrasmedbio.2003.11.008 11. Dattari TK, Whitesides TE Jr, Heller JG, Goodrich AC, McCarey BE, Hutton WC. Nicotine on the revascularisation of bone graft: An experimental study in rabbits. *Spine (Phila Pa 1976).* 1994;19(8):904-11. doi:10.1097/00007632-199404150-00007 12. Cook SD, Ryaby JP, McCabe J, Frey JJ, Heckman JD, Kristiansen TK. Acceleration of tibia and distal radius fracture healing in patients who smoke. *Clin Orthop Rel Res.* 1997;(337):198-207. doi:10.1097/00003086-199704000-00022 13. Naruse K, Sekiya H, Harada Y, et al. Prolonged endochondral bone healing in senescence is shortened by low-intensity pulsed ultrasound in a manner dependent on cox-2. *Ultrasound Med Biol.* 2010;36(7):1098-108. doi:10.1016/j.ultrasmedbio.2010.04.011 14. Coords M, Breitbart E, Paglia D, et al. The effects of low-intensity pulsed ultrasound upon diabetic fracture healing. *J Orthop Res.* 2011;29(2):181-8. doi:10.1002/jor.21223 15. Naik AA, Xie C, Zusick MJ, et al. Reduced COX-2 expression in aged mice is associated with impaired fracture healing. *J Bone Miner Res.* 2009;24(2):251-64. doi:10.1359/jbmr.081002 16. Tang CH, Yang RS, Huang TH, et al. Ultrasound stimulates cyclooxygenase-2 expression and increases bone formation through integrin, focal adhesion kinase, phosphatidylinositol 3-kinase, and Akt pathway in osteoblasts. *Mol Pharmacol.* 2006;69(6):2047-57. doi:10.1124/mol.105.022160 17. Heckman JD, Sarason-Jahn J. The economics of treating tibia fractures. The cost of delayed unions. *Bull Hosp Jt Dis.* 1997;56(1):63-72. 18. Zura R, Mehta S, Della Rocca GJ, Jones J, Steen RG. A cohort study of 4,190 patients treated with low-intensity pulsed ultrasound (LIPUS): findings in the elderly versus all patients. *BMC Musculoskeletal Disorders.* 2015;16:45. doi:10.1186/s12891-015-0498-1

Summary of Indications for Use:

EXOGEN is indicated for the non-invasive treatment of osseous defects (excluding vertebra and skull) that includes the treatment of delayed unions, nonunions,⁷ stress fractures and joint fusion. EXOGEN is also indicated for the acceleration of fresh fracture heal time, repair following osteotomy, repair in bone transport procedures and repair in distraction osteogenesis procedures. *A nonunion is considered to be established when the fracture site shows no visibly progressive signs of healing. There are no known contraindications for the EXOGEN device. Safety and effectiveness have not been established for individuals lacking skeletal maturity, pregnant or nursing women, patients with cardiac pacemakers, on fractures due to bone cancer, or on patients with poor blood circulation or clotting problems. Some patients may be sensitive to the ultrasound gel. Full prescribing information can be found in product labeling, or at EXOGEN.com

Bioventus Coöperatief U.A. Customer Care
Taurusavenue 31 T (UK): 0800 05 16 384 (toll free)
2132 LS Hoofddorp T (IR): 1800 552 197 (toll free)
The Netherlands E: customercare-international@bioventusglobal.com

Bioventus, the Bioventus logo and EXOGEN are registered trademarks of Bioventus LLC.

© 2021 Bioventus LLC SMK-000801B 11/21